

JacketJoy User Guide

Note: Before beginning this guide, please install the patches as shown in the patch placement guide




Tip: The placement shown here is right-sleeve first, mirror the placements for left-sleeve first.

1

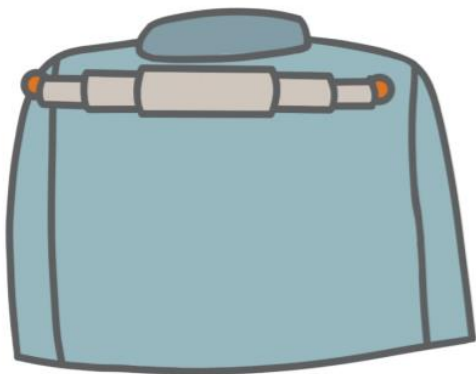

Attach the JacketJoy bar to the jacket by lining up the magnets in the bar and patches.


2

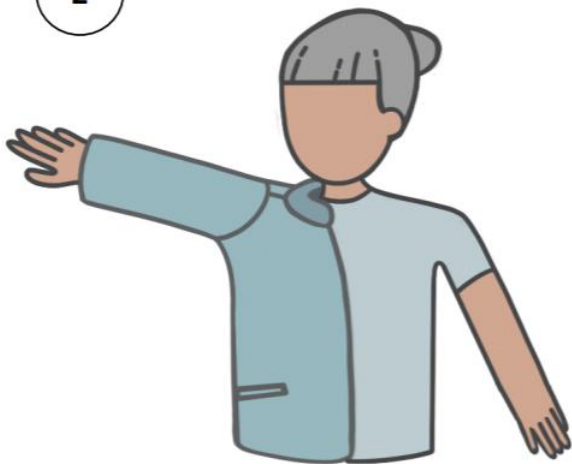

Insert your first arm (your “trouble” side or the side of your injury). Adjust the first-arm-side of the jacket so that it lays flat against your body.


3

Tip: This is the trickiest step, but it's key!
It may take a little practice to get right.

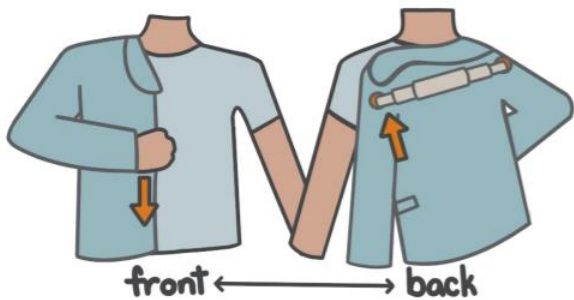

Grab the first-arm-side zipper of the coat with your first hand, a few inches below the collar. Wiggle this hand up and down to get a feel for how it moves the JacketJoy bar. Adjust the second sleeve to a comfortable position.

4

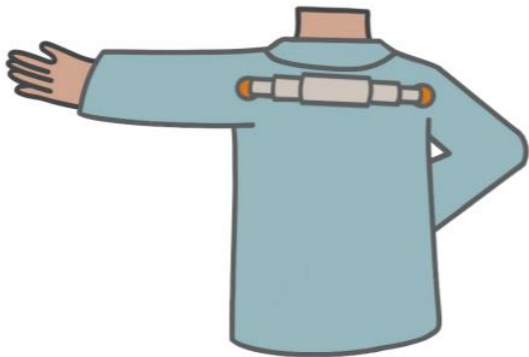


While still holding the jacket with your first hand, insert your second arm. This should be made easier by the positioning done in the previous step. If not, go back.

5



Hunch your shoulders slightly to “pop” the bar off your back.

6

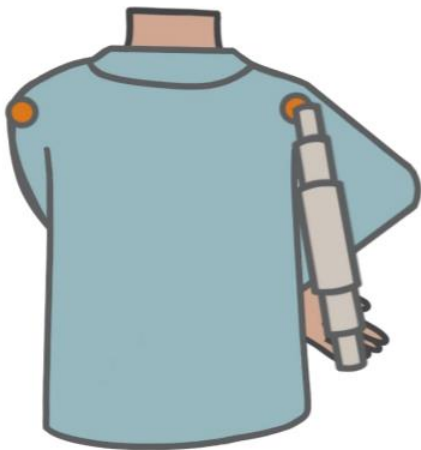

Retrieve the bar.